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TESTAmed[®]
DIABETES CARE

TESTAmed GlucoCheck ADVANCE
**GLUCOSE
CONTROL SOLUTION**

Intended Use

TESTAmed GlucoCheck ADVANCE Control Solution is intended for use with blood glucose monitoring systems **TESTAmed GlucoCheck ADVANCE**. It is used to check that the meter and test strips are working together properly. It is important to do control solution tests routinely to ensure getting accurate results.

Summary and Explanation

TESTAmed GlucoCheck ADVANCE Control Solution contains a known amount of glucose that reacts with **TESTAmed GlucoCheck ADVANCE** blood glucose test strips.

Operation of the control solution test is similar to blood glucose test except that you use **TESTAmed GlucoCheck ADVANCE** control solution instead of a drop of blood. The control solutions should provide results within an expected range printed on the test strip vial label.

IMPORTANT

The control range may change with each new vial of test strips. Always use the control range on the label of your current vial of test strips.

When should the control solution test be performed?

- When you first get your meter.
- When you begin using a new vial of test strips.
- Whenever you suspect that the meter or test strips are not working properly.
- When your blood glucose test results are inconsistent with how you feel, or when you think your results are inaccurate.
- If you drop the meter.

IMPORTANT

The control solution range printed on the test strip vial is for control solution only. It is used to test meter and test strip performance. **It is not a recommended range for your blood glucose level.**

Chemistry

TESTAmed GlucoCheck ADVANCE Control Solution is a red solution that contains less than 0.3 % D-glucose. The solution does not contain any human-derived or biological materials.

Ingredient	Percent
Active ingredient: D-glucose	0,019 % - 0,25 %
Inert ingredients	>99,75%

Storage and Handling

- For in vitro diagnostic use only.
- Use only **TESTAmed GlucoCheck ADVANCE** Control Solutions with **TESTAmed GlucoCheck ADVANCE** blood glucose monitoring systems.
- Check the expiration date on the vial. Do not use if expired.
- Tightly close the control solution and store at room temperature between 2 °C – 30 °C (36 °F – 86 °F). Do not freeze.
- Use only for 3 months after first opening. Record the discard date (date opened plus 3 months) on the control solution vial. Discard after 3 months.

Test Procedure

To do a test with **TESTAmed GlucoCheck ADVANCE** Control Solution, follow the instructions printed in the Owner's Manual that came with the system.

Accurate results depend upon following instructions carefully. Follow below technical pointers whenever you test with **TESTAmed GlucoCheck ADVANCE** Control Solution:

1. It is recommended that the meter, control solution, and test strips come to room temperature (20 °C – 25 °C / 68 °F – 77 °F) before testing.
2. Shake the control solution well before using.
3. Discard the first drop of control solution you squeeze out.
4. Squeeze out another drop and touch with the absorbent hole of a test strip.
5. Wipe off the dispenser tip to keep it clean.

Expected Results

At room temperature, **TESTAmed GlucoCheck ADVANCE** Control Solution test results should fall within the expected range printed on the test strip vial label.

If test results fall outside the expected range printed on the test strip vial, please check the troubleshooting guide in Owner's Manual and repeat the test. Results that fall outside the expected range may indicate:

- An error in the way you are doing the test.
- Expired test strips and control solutions.
- A problem with the meter.
- Control solution that is too warm or too cool.
- Failure to shake the control solution vial vigorously.

If the repeat result is still outside the range, the system may not be working properly. Do not use the system to test your blood. Please contact the **TESTAmed** Customer Service, if you have any questions.

Availability

TESTAmed GlucoCheck ADVANCE Control Solution is available in pharmacies or via Internet: www.testamed.de
For further information please contact the **TESTAmed** Customer Service.

OUR COMMITMENT TO YOU

Our goal is to provide you with quality healthcare products and dedicated customer service. If you are not fully satisfied with this product, or if you have questions about the use of any **TESTAmed GlucoCheck ADVANCE** product, please contact the **TESTAmed** Customer Service Hotline For service.

TESTAmed Customer Service Hotline: 0800 TREND11 (0800-87363-11)
www.testamed.de

Importer: Trend Pharma GmbH, Binger Straße 80, 56154 Boppard-Bad Salzig, Germany
Distributor: Sebapharma GmbH & Co. KG, P.O. Box 12 20, 56156 Boppard, Germany

 ForaCare Suisse AG
Neugasse 55, 9000 St. Gallen, Switzerland
www.foracare.ch

 MedNet EC-REP CmbH
Borkstraße 10, 48163 Münster, Germany

 Zur Selbstkontrolle.
Kendi kendine test icindir.
For self-testing.
  

313-000000-278